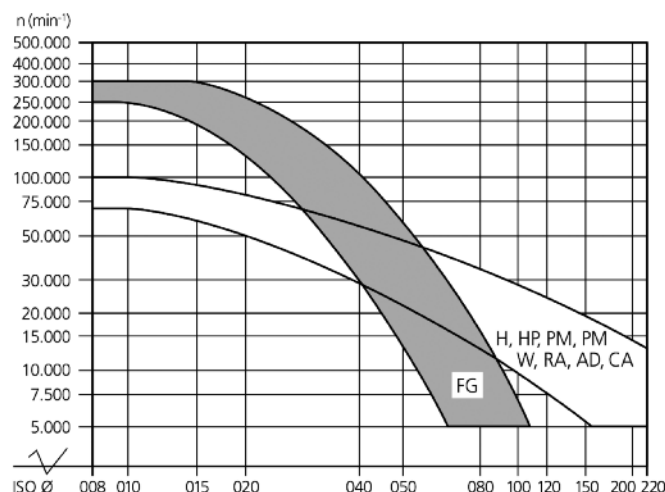
dental diamant

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dental diamond

Directions for use of rotary diamond instruments (FG and right angle type)

The following instructions enable the best preparation results to be achieved by handling and using the instruments carefully. Of paramount concern are conservation of the tooth substrate and hygiene requirements.

- The separate information and instructions for cleaning and sterilising electroplated dental diamond burs with FG or right angle shanks apply with regard to cleaning/sterilisation.
- Instruments are to be selected according to their diameter and shape so as to avoid damaging adjacent anatomical structures.
- Preference should be given to rounded shapes.
- The indication is limited for instruments with coarse grit; as finish grinding is usually required.
- It is advisable to use drill holders or, for longer storage, the original packaging.
- Make sure the instruments are firmly secured when using them in a nozzle or handpiece.
- The speeds given in the diagram, taking into account the head diameter, are not to be exceeded. Bring the instruments up to working speed away from the preparation area.
- Spray cooling with a minimum flow of 50 ml/min is to be used to protect the dental pulp.
- Additional cooling is required when using instruments longer than 21 mm or with a head diameter larger than 2 mm.
- The force must not exceed 2 N (200 p); a greater force will increase the thermal load, but will not remove the material more quickly.
- Worn, damaged or non-concentrically running instruments must be disposed of.
- The instruments are to be placed in a "drill bath" between being used and sterilised (follow the manufacturer's instructions).
- A hard brush can be used for mechanical cleaning of the grinding layer.

- Diamond instruments are resistant to corrosion; however, they must not be exposed to acids.

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>>>

Information and instructions for cleaning and sterilising electroplated dental diamond burs with FG or right angle shanks

The following information serves solely as recommendations or instructions for handling the aforementioned electroplated diamond tools. They conform with the following documents:

- Robert Koch Institute: "Hygiene requirements for the preparation of medical products", Federal Health Gazette 2012 - 55:1244-1310
- Robert Koch Institute: "Prevention of infection in dentistry - hygiene requirements", Federal Health Gazette 2006 – 19:375 - 394

1. Products for which these recommendations are applicable

Dental diamond burs from fis with an FG or right angle shank can generally be reused. This requires, however, that the tools are prepared prior to reuse just as before they are used for the first time. Previously used burs with an FG or right angle shank must be cleaned, checked and sterilised before they are used again. This applies similarly to brand new burs. The latter must be sterilised before being used for the first time.

2. Qualification of persons who prepare these products

This work should be performed solely by persons who have acquired and demonstrated the necessary knowledge and skills through instruction, training or practical experience.

3. Training

These persons must first be trained in the use of new products or following procedural upgrades and changes.

4. Related documents

It should be ensured that the recommendations issued by the Robert Koch Institute and the Federal Institute for Pharmaceuticals and Medical Products, "Hygiene requirements for the preparation of medical products", are complied with taking into consideration the special dentistry requirements.

5. Transport and storage

The cleaning and disinfection of abrasive layers are subject to special requirements due to their rough surfaces. The tools must always be stored and transported so that the high-precision manufactured shafts are not damaged by the abrasive layers. To avoid hardening of grinding residues, the tools should be placed in a suitable cleaning bath or disinfection medium ("drill bath") immediately after use (less than 60 minutes).

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>>>

6. Cleaning and disinfection

6.1. The surfaces of the dental burs must first be thoroughly rinsed under cold running water and cleaned with a hard brush (do not use a wire brush) to remove both loose abrasion particles as well as residues of cleaning or disinfecting agents. The brush should then be cleaned and disinfected.

6.2. The tools should then be cleaned in an ultrasonic bath at a temperature of 65 °C for approximately 1 to 3 minutes. The manufacturer's instructions must be observed. During the ultrasonic treatment, the grinding layers should not come into contact with the shafts of other tools to avoid damage.

6.3. After cleaning in the ultrasonic bath, the grinding layers must be examined under a magnifying glass with at least 6x magnification. Steps 6.1 and 6.2. must be repeated to remove any adhesions still present on the layers. The tools must be disposed of if the layers are worn, have defects or the adhesive particles can no longer be removed. This also applies in the event of visible damage on the shaft or layer.

6.3. In accordance with RKI guidelines, preference should be given to an automatic cleaning/disinfecting procedure instead of a manual procedure to assure a high degree of reproducibility. The manufacturer's specifications are to be observed. The "Vario TD AD" procedure is described below using a device such as the Miele G 7835 CD:

- 2 pre-rinsings, followed by draining of the washing water
- Cleaning with tap water at 55 °C for 5 minutes
- Cleaning agent, e.g. "Mediclean Neodisher", 0.5%
- Washing water is drained, followed by rinsing with deionised water
- Thermal disinfection at between 90 °C and 94 °C for at least 10 minutes - neutralisation with deionized water using "Neodisher Z", 0.1%
- Drain water - device drying cycle at 50 °C for 30 minutes

6.4. All usable burs with a diamond layer must then undergo steam sterilisation in a fractionated vacuum process at 134 °C. The device should be designed pursuant to EN 285, validated processes.

- Fractionated vacuum (4-fold)
- Sterilisation temperature 134 °C
- Dwell time - 5 minutes (full cycle)
- Drying time - 10 minutes

The tools should subsequently be stored so that they are protected against contamination.

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Concluding remarks

Applicable national regulations are to be observed when reprocessing used diamond tools.

On account of their materials and workmanship, tools manufactured by fis are generally suitable for reprocessing using the described procedure. Users are responsible for reprocessing the tools in compliance with this recommendation and the operating instructions and instructions for use from the manufacturers who supply the appropriate devices and materials. Users are also responsible for ensuring that the reprocessing attains the intended results with the described equipment and qualified personnel.

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On our packages the below listed pictographs will appear to indicate the specific use :

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