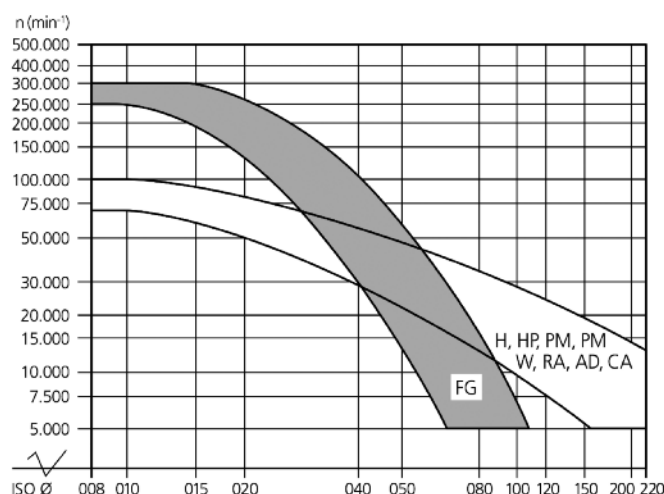
dental diamant

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dental diamond

Directions for use of rotary diamond instruments (FG and right angle type)

The following instructions enable the best preparation results to be achieved by handling and using the instruments carefully. Of paramount concern are conservation of the tooth substrate and hygiene requirements.

- The separate information and instructions for cleaning and sterilising electroplated dental diamond burs with FG or right angle shanks apply with regard to cleaning/sterilisation.
- Instruments are to be selected according to their diameter and shape so as to avoid damaging adjacent anatomical structures.
- Preference should be given to rounded shapes.
- The indication is limited for instruments with coarse grit; as finish grinding is usually required.
- It is advisable to use drill holders or, for longer storage, the original packaging.
- Make sure the instruments are firmly secured when using them in a nozzle or handpiece.
- The speeds given in the diagram, taking into account the head diameter, are not to be exceeded. Bring the instruments up to working speed away from the preparation area.
- Spray cooling with a minimum flow of 50 ml/min is to be used to protect the dental pulp.
- Additional cooling is required when using instruments longer than 21 mm or with a head diameter larger than 2 mm.
- The force must not exceed 2 N (200 p); a greater force will increase the thermal load, but will not remove the material more quickly.
- Worn, damaged or non-concentrically running instruments must be disposed of.
- The instruments are to be placed in a "drill bath" between being used and sterilised (follow the manufacturer's instructions).
- A hard brush can be used for mechanical cleaning of the grinding layer.

- Diamond instruments are resistant to corrosion; however, they must not be exposed to acids.

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>>>

Information and instructions for cleaning and sterilizing electroplated dental diamond burs with FG or right-angled shanks.

Here you will find recommendations and instructions for handling the above-mentioned electroplated diamond tools.

1. Products:
In general, fis dental diamond burs with FG or right-angled shanks can be reused, provided they are reprocessed before second use. Used burs with FG or right-angled shanks must be cleaned, inspected and sterilized before their next use. New burs from the factory must be sterilized before first use.

2. Qualification of staff reprocessing these products
This work should only be carried out by someone who has gained the necessary knowledge and skills through training, qualifications or experience.

3. Training
When using new products or updating/changing procedures, additional training is required.

4. Further documents
It must be ensured that the recommendations of the Robert Koch Institute and the German Federal Institute for Pharmaceuticals and Medical Products in "Requirements for hygiene when reprocessing medical products" are followed, especially those concerning dental health.

5. Transport and storage
The products are supplied unsterilized. As the burs have a rough surface, cleaning and disinfecting requirements are particularly high. The tools must be stored and transported in such a way as to avoid damage to the high-precision shanks by the grinding layer. To avoid grinding residue hardening, the tools must be put into a suitable cleaning bath or disinfectant (drill bath) directly after use (within 60 minutes).

6. Cleaning and disinfection

6.1 Pre-treatment at place of use
First, the tools should be cleaned with a lint-free disposable cloth.

6.2. Disposal and transport to area of reprocessing
We recommend putting the tools directly into a cleaning bath or disinfectant (drill bath) after use and transporting them in the fluid to the area of reprocessing.

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>>>

6.3. Manual cleaning and disinfection

- The grinding layers of the dental bur should be rinsed thoroughly under running cold water and cleaned with a hard brush (not a metal wire brush) to remove loose particles as well as residue from cleaning agents or disinfectant. The brush should then be cleaned and disinfected.

- Clean the burs in an ultrasonic bath (0.5% neodisher® MediClean forte) at a temperature of <math><40^{\circ}</math> Celsius and a frequency of 35kHz for 10 minutes. Please refer to the recommendations of the bath and cleaning agent manufacturers.

- To avoid damage while cleaning, the burs should not come into contact with the shanks of other tools.

- Clean products thoroughly with a soft brush and rinse under running drinking water (>1 minute).

- Dry manually using a lint-free disposable cloth.

- Dip products into a disinfectant listed by the RKI or the Association for Applied Hygiene (VAH). Follow the manufacturer's instructions. Ensure that the disinfectant reaches all areas of the product.

- The process is validated with the following disinfectant: 3% Korsolex Plus, 15 minutes.

- Rinse the product with de-ionised water for at least 1 minute (or as instructed by manufacturer).

- Dry with a lint-free disposable cloth. We recommend blowing the grinder with sterile, oil-free compressed air.

- Do not heat product to more than 140°C.

- After cleaning in an ultrasonic bath, inspect the bur with a magnifying glass with at least 6x magnification. If any particles or residue are still on the bur, the above steps must be repeated. If the grinding layer is worn, damaged or particles or residue cannot be removed, dispose of the tool. This also applies to any damage on the shank or bur seen by the naked eye.

6.4. Automatic cleaning and disinfection

Following guidelines from the RKI automatic cleaning should be the preferred method to ensure high reproducibility. Follow the manufacturer's instructions.

- The grinding layers should be rinsed thoroughly under running cold water and cleaned with a hard brush (not a metal wire brush) in order to remove loose particles and residue from cleaning agents and disinfectant.

- Place carefully into a disinfectant.

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The „Vario TD AD“ process of a disinfectant such as the Miele G 7835 CD is described below:

- 2 x rinse for 1 and 3 minutes with tap water. Water is then drained.
- Clean with tap water and alkaline cleaner (e.g. „neodisher® Mediclean Dental“ 0.5%) at 55° C for 10 minutes.
- Neutralization with tap water and 0.1% „neodisher® Z“.
- Drainage of water and rinse with iodised water for thermal disinfection at 90°C to 94°C for at least 10 minutes
- Drying cycle of 30 minutes

The product may not be heated to more than 140°C

To avoid damage while cleaning automatically the burs should not come into contact with the shanks of other tools. After cleaning in a disinfectant inspect the bur with a magnifying glass with at least 6x magnification. If any particles or residue are found on the bur, the above steps must be repeated. If the grinding layer is worn, damaged or particles or residue cannot be removed, dispose of the tools. This also applies to any damage on the shank or bur seen by the naked eye.

6.5. Sterilization

All usable burs with diamond layers should be packed in suitable sterile packaging (ISO 11607) which has been steam sterilized in a fractional vacuum process at 134°C. The device should comply with EN 285 / EN ISO 17665-1 validated processes.

- Fractional vacuum (4-fold)
- Sterilization temperature 134° C
- Dwell time 5 minutes (full cycle)
- Drying time 20 minutes

When stored, the tools must be protected from contamination.

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7. Final note

The regulations of different countries should be considered when reprocessing diamond tools.

The materials and their characteristics on tools manufactured by fis are generally suitable for reprocessing as described above. Those using the tools are responsible for reprocessing following these recommendations and the manufacturer's instructions of devices and materials. They are also responsible for ensuring that reprocessing is carried out with the equipment described here and by qualified staff to achieve the intended results.

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